

MOTOR REPLACEMENT FOR #62 SUPER SUCKER

1. Remove units cordset from its electrical connection
2. Remove top of unit and clean all filters.
3. After cleaning filters, place unit on sturdy, flat table top on its back
4. Remove the four rubber feet from the bottom of the 62
5. After removing the rubber feet, remove the bottom of the 62. The charcoal filter is located in the bottom plate covering the bottom of the dust collector. Check this filter for cleanliness, and replace if necessary (if over one year old, replace).
6. The two suction motors are now accessible. Remove the motor(s) which is being replaced. Make certain to label all wires which are connected to the "old" motor. These should be labeled BLACK (L-1), WHITE (L-2) and GROUND. Attach the BLACK (L-1) wire to the black wire on the new motor. Attach the WHITE - neutral wire to the small vertical terminal located on the side of the suction motor next to the motor brush holder. This spade terminal does not have a wire connected to it. Connect the GROUND wire to the brass terminal on the side of the motor.
7. Replace the new motor into the location from which the "old" motor was removed. Secure in place.
8. Replace the bottom of the 62 and secure it in place with the four rubber Grommets previously removed. Your unit is now ready for operation.

Should you have questions, please contact your distributor or Handler Mfg. Co. Inc at:

HANDLER MFG. CO. INC 612 NORTH AVENUE, EAST WESTFIELD, N.J. 07090
908-233-7796 1-800-274-2635 Fax: 908-233-7796

